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Stand der Information: 11/2016
 Made in Germany

Congratulations on purchasing this RÖWO® Cold/Warm Compress. You have acquired a high-quality medical product for cryoapplication and heat treatment. These directions for use are an integral part of the product. They contain important information relating to safety, usage and disposal. Familiarise yourself with all usage and safety instructions before use. Only use the product as described and for the stated fields of application. Hand over all documents when passing the product on to third parties.

Key to symbols:

- Read and store these directions for use carefully.
- Always adhere to the stated warning information.
- Do not heat the product to more than 50°C.
- Product for heating in a microwave oven
- Product suitable for freezers
- Store the product out of the reach of children. Do not use for children under 3 years old; older children must be supervised during use.

Directions for use for cryoapplication:

e.g., headaches, migraines, itchiness, nosebleeds, haematomas, fevers, insect bites, toothaches, bruises, sprains and burns.

- 1.** Place in fridge or freezer for approx. 1 h

- 2.** Put cold pack into the enclosed sleeve.

- 3.** Place cold pack on the area to be treated.

Warnings for cryoapplication:

For external use only. Danger of localized frostbite if used without fabric sleeve or cloth.

Do not use:

- for children under 3 years old;
- in the case of peripheral circulatory disorders;
- in the case of Raynaud's Syndrome;
- in the case of limited temperature sensitivity;
- when using pain-killing drugs;
- on damaged skin; and
- on open wounds.

Consult a doctor before use in the case of circulatory complaints and other pre-existing conditions.

Store the product out of the reach of children. Children must be supervised during use

Directions for use for heat treatment:

e.g. for tension, lumbago, rheumatism, tendonitis, periostitis, fibromyalgia, muscle aches or menstrual cramps. The product is available in different sizes. The size is stated on the product. The different product versions have different heating times:

- 1.** Heat in the microwave oven at 180 W (defrost):

Size	10 x 16 cm 2 min.	13 x 28 cm 4 min.
	15 x 28 cm 4 min.	20 x 30 cm 4 min.

- 2.** Knead the heat pack.

- 3.** Heat again in the microwave oven at 180 W (defrost):

Size	10 x 16 cm 1 min.	13 x 28 cm 2 min.
	15 x 28 cm 2 min.	20 x 30 cm 2 min.

Alternatively: Heat the heat pack to 50 °C in a water bath.

- 4.** Put the heat pack in the enclosed sleeve.

- 5.** Place the heat pack on the area to be treated.

Design differences between microwave ovens can lead to different heating results. You should therefore monitor the product during heating and stop the heating process immediately if the product inflates. Higher powers or temperatures can damage the product. Carefully remove the cold/warm compress from the microwave oven, knead thoroughly and carefully test the temperature with the back of your hand. If it is too hot, leave it to cool. If the desired temperature has not yet been reached, continue to heat and test the temperature at 15-second intervals.

Warnings for heat treatment:

For external use only. Danger of burns if the product is overheated!

Inspect before use and do not use the product if damaged; instead, dispose of it as household waste. Do not heat in the oven. Protect from open flames.

Do not use:

- for children under 3 years old;
- on damaged skin;
- on open wounds;
- with inflammatory processes;
- in the case of limited temperature sensitivity; and
- when using pain-killing drugs.

Consult a doctor before use in the case of circulatory complaints and other pre-existing conditions.

Store the product out of the reach of children. Children must be supervised during use.

Cleaning the cold/warm compress:

After use, clean under running, hand-hot water using a mild soap. Do not use solvent-containing cleaning agents, such as disinfectants. Dry with a soft cloth or leave to air-dry.

Guarantee

The product has been produced in accordance with strict quality guidelines and has been tested thoroughly. The guarantee applies to material or manufacturing defects. The guarantee does not cover normal wear and tear or improper use or damage. All instructions listed in the directions for use must be observed precisely in order to constitute proper use. The product is intended for private use only – i.e., not for commercial use. The gel inside the cold/warm compress is non-toxic. In case of contact, wash off with soap and water.

Disposal

A damaged cold/warm compress can be disposed of as normal household waste.

LOT Lot number: see product

Cold/Warm Compress Product Group 0203

Manufacturer:
 Sporto-med GmbH
 Willy-Brandt-Ring 12
 D-67592 Flörsheim-Dalsheim · Germany
 info@sporto-med.de

Information as at: 11/2016
 Made in Germany