RAPID

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RAPID

AGAINST CORNS

this results in a milky white protective membrane, which should not be removed. Repeat 2x daily. The corn is usually easily removed after 5-7 days. If any residue, the so-called root of the corn, remains once the white membrane is removed, repeat the procedure. Treatment of calluses follows in the same manner.

WHEN MAY RAPID NOT BE USED?

- In the event of hypersensitivity to one of the product's components.
- Skin lesions not clearly identified as corns.

WARNINGS AND PRECAUTIONS

Application on larger skin areas should be avoided, especially in small children and persons with severe circulatory disorders (e.g. diabetics).

WHAT ARE THE POSSIBLE SIDE EFFECTS?

RAPID may cause skin irritations on healthy skin. To avoid this, the healthy skin surrounding the corn or callus should be covered with an oily skin cream or Vaseline - see instructions for use above. In the event of prolonged irritation (severe burning or itching, inflammation) discontinue treatment and seek medical advice if necessary.

WARNINGS

RAPID is highly flammable, therefore do not use near an open fire or while smoking. Always close the bottle tightly. Store at room temperature. Keep this medical device away from children and avoid contact with the eyes and mucosa. Do not use this package after the imprinted expiration date.

COMPOSITION

Colloidion, salicylic acid, lactic acid, acetone, castor oil.

BANO Healthcare GmbH
Annagasse 17, 2500 Baden
Austria, www.bano.at

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Physical effect through peeling

INSTRUCTIONS FOR USE

Before using this medical device, read the entire patient information leaflet carefully as it contains important information. Always use this medical device exactly as described in this patient information leaflet or as instructed by your doctor or pharmacist.

WHEN IS RAPID USED?

RAPID is used for the quick removal of corns and calluses. This solution softens excessively callused skin for easy removal after 5-7 days.

WHAT CAUSES CORNS AND CALLUSES?

Corns and calluses can be caused by one-sided, excessive pressure on the skin and comprise localised thickening of callused skin. Tight shoes or malpositioned feet are usually the cause of corns. They most frequently occur on the 4th and 5th toe or on the underside of the foot and can sometimes cause severe pain.

HOW IS RAPID USED?

The corn or callus is brushed with RAPID in the mornings and evenings. In the evening, the application should be preceded by a warm foot bath, softening the skin in warm water for about five minutes and then drying it well. If possible, carefully scrape off the callused skin with a pumice stone or a file. Make sure that you do not injure the skin. Completely cover the healthy skin around the treated area with oily skin cream or Vaseline and brush the corn with the solution using the enclosed brush;

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