Evolsin

Evolsin® Hämorrhoiden Gel

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CE 0481

Evolsin® Hemorrhoid Gel

- Treatment of external hemorrhoids
- Relief of hemorrhoid symptoms such as burning, itching and pain caused by hemorrhoids
- Relief of local irritation, fissures and external discomfort

General advise:

Carefully read the instructions for use prior to using **Evolsin® Hemorrhoid Gel**, even if you have used this medical device before. The information may have been amended to take account of new research and findings in the time since you opened your last package. This medical device can be obtained without a prescription. In spite of this, take care when using **Evolsin® Hemorrhoid Gel** in order to obtain a good result. Keep these instructions for use in case you need to consult them again.

Application:

Evolsin® Hemorrhoid Gel is a medical device for the treatment of external hemorrhoids and other minor anorectal conditions such as anal itching and fissures. This protective gel reduces irritation, swelling, itching, pain and burning, and prevents contact of the irritated skin with aggravating liquid or stool from the rectum.

Dosage and use (for adults):

Squeeze the tube of **Evolsin® Hemorrhoid Gel** and apply a small amount on your finger. Apply the gel around the outside of your anus with your finger. Wash your hands thoroughly with warm water and soap after each application. Apply the gel to the affected area two to four times a day, preferably after a bowel movement, for 7-14 days.

How it works:

Evolsin® Hemorrhoid Gel forms a protective layer with excellent moisturizing and anti-inflammatory properties. The layer protects the area around the anus and restores the structural integrity of veins and normal blood flow to the area around the anus. It reduces inflammation and the risk of infection, not because it kills bacteria but because it prevents them from penetrating the skin. It helps to resolve a prolapsed Hemorrhoid before it grows large and produces a smoothing effect for the passage of hard stools.

Possible side effect(s):

Irritation could be a side effect.

Use during pregnancy and/or breastfeeding:

Hemorrhoid gel is not recommended in pregnant or breastfeeding woman due to a lack of adequate available human data on the active ingredients.

Warnings:

- Keep out of reach of young children;
- If you are suffering from hypersensitivity reactions, stop using Hemorrhoid Gel;
- Do not use after the expiry date stated on the packaging.

Composition:

Aqua, Glycerin, Betaine, Panthenol, Ammonium Acryloyldimethyltaurate/VP Copolymer, Laureth-9, Phenoxyethanol, Xanthan Gum, Allantoin, Ethylhexylglycerin, Sodium Carboxymethyl Betaglucon, Opuntia Ficus-Indica Stem Extract, Sodium Hyaluronate.

Content: 40 ml

Storage method:

Store at room temperature (5-25°C) in a dry and dark place in the original container and out of reach of young children.

Final date of use:

We advise against using **Evolsin® Hemorrhoid Gel** after the expiry date, which is stated on the packaging after 'expiry date'. **Evolsin® Hemorrhoid Gel** can be kept for 1 month after opening.

Medical device:

Hemorrhoid Gel is a class IIA medical device in accordance with the Medical Device Directive 93/42/EC. Date on which these instructions for use were last revised: December V2.2

Manufacturer:
PK Benelux BV
Vluchttoord 17, 5406 XP Uden, NL

Vertrieb:

Evolsin medical UG, Schlehenweg 7,
D-61267 Neu-Anspach; www.evolsin-medical.com

PZN 171 50459

CE 0481

237mm

150mm