DE: WUNDmed® Akupunkturpflaster (Cross-Tape / Gitter-Tape)

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EN: WUNDmed® Acupuncture Plaster (Cross-Tape)

The WUNDmed® Acupuncture Plaster, also known as Cross-Tape or Grid-Tape, is a grid-shaped plaster for external use without medicinal ingredients. It is applied to the skin as a supportive measure, particularly within the framework of traditional Asian methods such as Traditional Chinese Medicine (TCM). A therapeutic benefit has not yet been scientifically proven.

Product Features:

Grid-shaped plaster without medicinal substances
Breathable, sweat- and water-resistant
For external use
Comfortable to wear
Easy to apply
Suitable for everyday life, sports, and leisure

Application Instructions:

Ensure the skin is dry, clean, and free of oil before application
Shave heavily haired areas if necessary
The plaster can be applied to specific points on the body (e.g., so-called trigger points)
It can be worn for several days as long as it remains comfortable
Showering with the plaster is possible
Apply approximately 20–30 minutes before physical activity

Do Not Use If:

Known allergy to plaster adhesive
Open wounds or mucous membranes
Tumor diseases
Acute inflammations (e.g., erysipelas)
Thrombosis or thrombophlebitis
Severe heart failure
Irritated or sensitive skin

In case of skin reactions (e.g. itching, redness, blistering), remove the patch and consult a doctor if necessary.

Important Notes:

This product does not replace medical diagnosis or treatment
Seek medical advice for unclear, persistent, or worsening symptoms
Report serious incidents related to the product to the manufacturer and the competent authority

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